MicroPeak

PEAK FLOW METER

INSTRUCTIONS FOR USE

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PO Box 6 Rochester
Kent ME1 2AZ, UK
+44(0)1634 360044

Kieler Strasse 675
22527 Hamburg, Germany
+49(0)40 5770970

Medical

CE 0120

www.micropeak.co.uk

About your Peak Flow Meter

Your MicroPeak Peak Flow Meter will help you to monitor your asthma by measuring your Peak Expiratory Flow Rate (PEF). It should be used in conjunction with your doctor or practice nurse, who will tell you what to do if your PEF changes.

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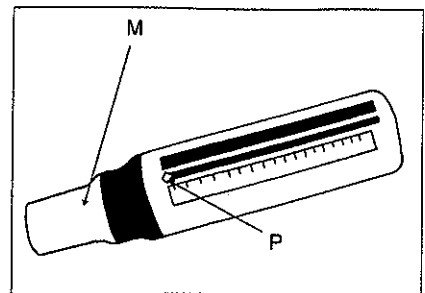
1

Insert the mouthpiece (M) into the meter, and move the pointer (P) to the zero end of the slot. Lightly hold the meter so that you do not obstruct the slot or the holes at the end.

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PG.1

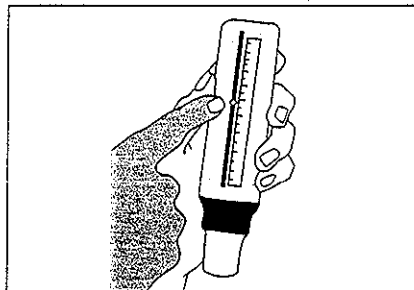
2

Standing up, or sitting up straight, take as deep a breath as possible and place the mouthpiece into your mouth, making a seal with your lips. Blow into the meter as fast and as hard as you can. Do not put your tongue into the mouthpiece.

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
3

Take the meter from your mouth and note the number opposite the pointer. Your PEF is the highest of three attempts. Repeat the procedure twice more and note the number each time. Enter the highest number of the three on the chart overleaf.

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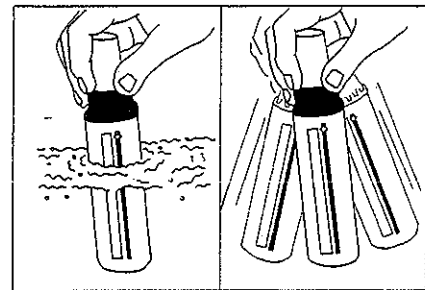
CLEANING Your meter should be cleaned from time to time. Immerse the meter in warm mild detergent solution for 2-3 minutes and agitate. Rinse in clean warm water, shake off excess water and allow to air dry. DO NOT SUBJECT THE METER TO BOILING WATER.

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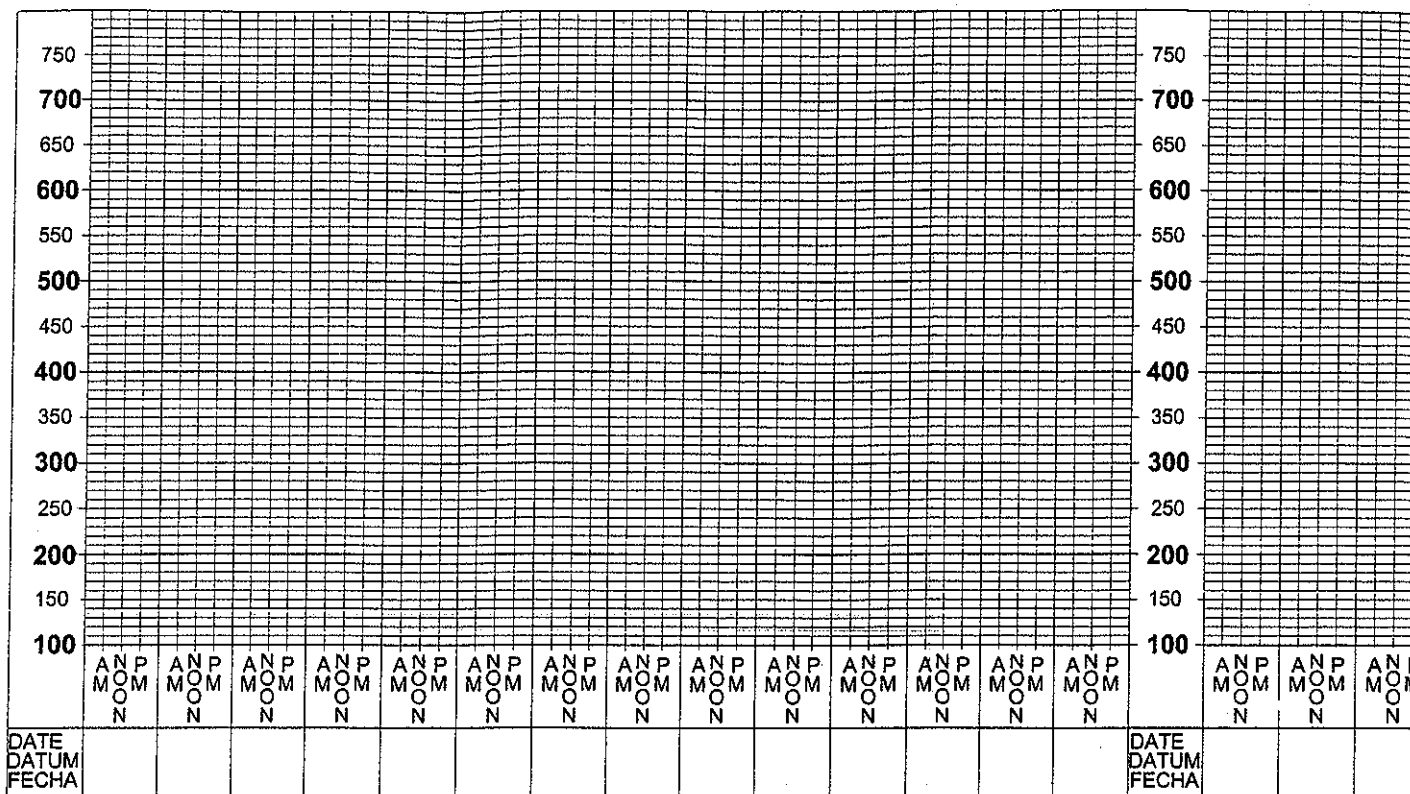PG.2

You should record your PEF two or three times a day as directed by your doctor or practice nurse. Plot your PEF on the chart (above) every day and write in the date. Join up the points as shown. Remember that only the highest value of your three attempts should be plotted. Your doctor will need to examine your PEF chart periodically.

For further information, and to download your personal asthma diary, visit our website: www.micropeak.co.uk

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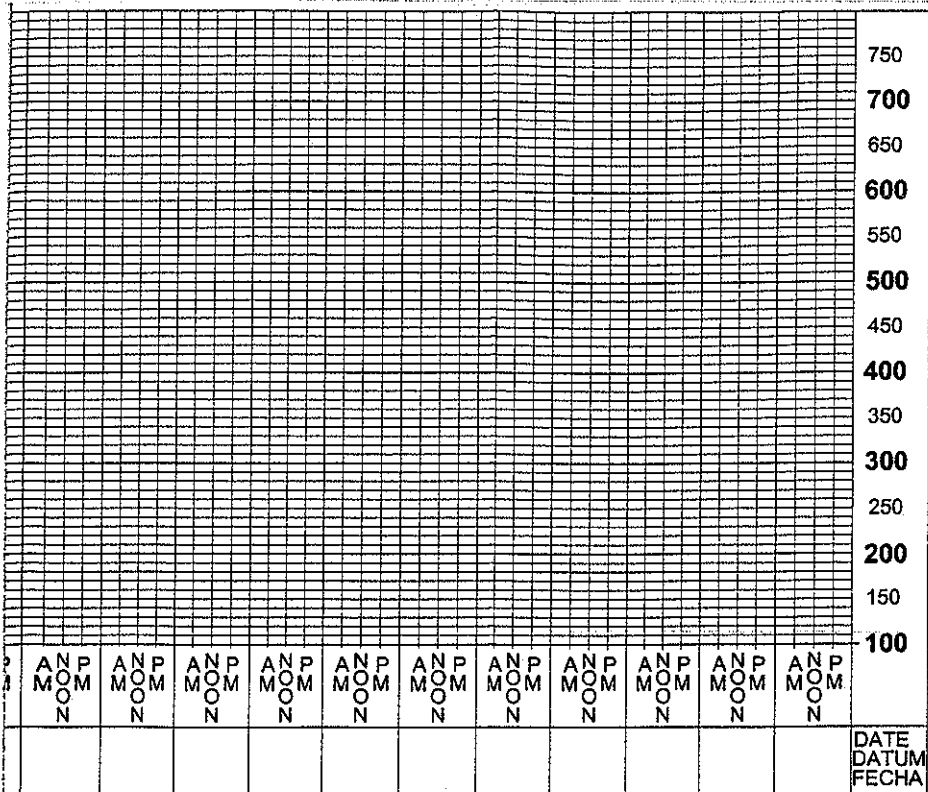
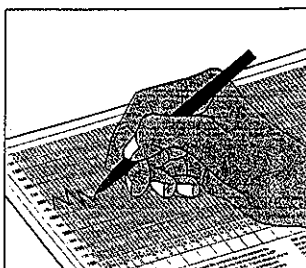
P M Evening
Soir
Abend
Tarde

TAKING CARE OF YOUR MicroPeak

Your MicroPeak Peak Flow Meter is manufactured from medical grade materials, which are water and corrosion resistant. Keep the meter in a dry, clean place and do not expose it to excessive heat. The detachable mouthpiece should be cleaned regularly in a mild disinfectant solution.

The MicroPeak meets all international standards and with careful use, will give accurate performance for at least three years.

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