
Checkme™ O2

Smart Wrist Pulse Oximeter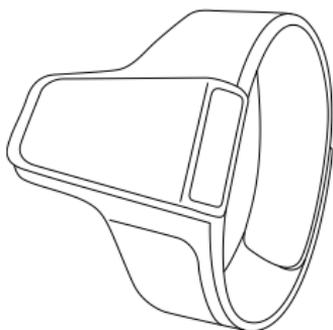

User Manual

viatom™
www.viatomtech.com

User's Manual

Contents

1. Introduction	4
2 Using the Device	6
3 Maintenance	11
4 Troubleshooting	11
5 Specifications	12
6 Electromagnetic Compatibility	13
1. Einführung	14
2. Verwenden des Geräts	17
3. Wartung	22
4.Fehlerbehebung	23
5.Spezifikationen	23

1. Introduction

1.1 Intended Use

The Oxiband Wrist Pulse Oximeter is intended to be used for measuring, displaying and storing of pulse oxygen saturation (SpO₂) and pulse rate in home or healthcare facilities environment.

Note:

The data and results provided by this device are for pre-check screening purpose only and cannot be directly used for diagnostic or treatment.

The data provided by the APP is not intended for diagnosis or treatment purpose, always consult your doctor for any health condition.

Warnings and Cautionary Advices

- Do not use this device during MRI examination.
- Do not use the device in a combustible environment (i.e., oxygen-enriched environment).
- Never submerge the device in water or other liquids. Do not clean the device with acetone or other volatile solutions.
- Do not drop this device or subject it to strong impact.
- Do not place this device in pressure vessels or gas sterilization device.
- Do not dismantle the device, as this could cause damage or malfunctions or impede the operation of the device.
- This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who has responsibility for their safety or they receive instructions from this person on how to use the device.
- Do not store the device in the following locations: locations in which the device is exposed to direct sunlight, high

temperatures or levels of moisture, or heavy contamination; locations near to sources of water or fire; or locations that are subject to strong electromagnetic influences.

- Vital signs measurements, such as those taken with this device, cannot identify all diseases. Regardless of the measurement taken using this device, you should consult your doctor immediately if you experience symptoms that could indicate acute disease.
- Do not self-diagnose or self-medicate on the basis of this device without consulting your doctor. In particular, do not start taking any new medication or change the type and/or dosage of any existing medication without prior approval.

1.2 Guide to Symbols

Symbol	Description
	Type BF-Applied Part
	Manufacturer
	European authorized representative
CE0197	CE Marking indicating conformance to EC directive No. 93/42/EEC concerning medical devices.
	MRI unsafe. Presents hazards in all MR environments as device contains strongly ferromagnetic materials.
	Indicate separate collection for electrical and electronic equipment (WEEE).
IP22	Protected against spraying water and against access to hazardous parts with a tool, per IEC 60529.
	Follow Instructions for Use.
	Warning and Caution!
SN	Serial number

No alarm system.

1.3 Unpacking

Main Unit × 1

Ring Sensor × 1

Cable × 1

User Manual × 1

2 Using the Device

2.1 Download App

App name: **ViHealth**

iOS: App Store

Android: Google Play

2.2 Charging

Charge the battery before using.

Connect the device to USB of computer or USB charging adapter.

After fully charged, the device will power off automatically.

2.3 POWER ON/OFF

POWER ON:

Press the button for 1 second to turn on the device.


POWER OFF:

Recommended way: device turns off automatically in 2 minutes if no measurement, no operation, without App connection.


Manually: press the button for about 2 seconds to turn off.

2.4 Typical steps

START. Charge the battery. Wear the device and the Ring Sensor, press the button to power on. Go to sleep.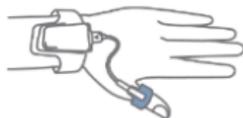

STOP. Next morning, take off the Ring Sensor (and the device), the recording will be over after the countdown.

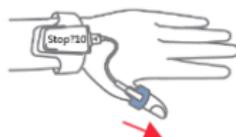DATA SYNC. Run App to sync data.

2.5 Start working

- 1) Connect the sensor cable to the device.
- 2) Wear the device on the left wrist; wear the sensor on the thumb. **If it is too tight, try another finger. Avoid being loose.**
- 3) Turn on the device. After a few seconds, the device will begin to work.
(If the working time is less than 1 minute, the data will not be saved)

Note:

- *The device can also work as a clock and pedometer without ring sensor, and the steps will restart counting after the ring sensor plugged in.*
- *Please avoid excessive motion for the sensed finger during recording and avoid any strong ambient light condition.*

2.6 Stop working & sync data

Take off the sensor, the countdown will begin.

During the countdown, if you wear the sensor again, the record will be resumed.

After the countdown, the data will be ready for sync.

Sync data:

- 1) After the countdown, Run App to sync data;
- 2) Or next time after you turn on the device, run App to sync.

Note: The device can store maximum 4 records. The oldest record will be overwritten when the 5th record is coming in.

2.7 Display

Time, Remaining battery capacity

Sleep Mode

Living Mode

SPO2

Blood oxygen saturation

Pulse rate in Sleep Mode

Pulse rate in Living Mode

Wear the sensor

Steps

O2 Score, SpO₂ drop times

During measurement, press the button, you can switch different displays.

2.8 Operation Mode Navigation

The device has two operation modes: **Sleep Mode** and **Living Mode**.

Turn on the device, during the icon blinking, press the button to switch between two modes.

	Sleep Mode	Living Mode
Parameters	SpO ₂ , PR, Motion	SpO ₂ , PR, Steps
Screen	Activated by button	Always on
The Sensor Vibrates	When SpO ₂ is lower than threshold set	When SpO ₂ is lower than threshold set; Or when step goal is met
Bluetooth	On when the screen lights up	Always on
Report	Sleep report	Living report

2.9 Bluetooth Connection

The device Bluetooth will be enabled automatically only when the **screen lights up**.

To establish a Bluetooth connection,

- 1) **Ensure the device screen is on to keep the device Bluetooth enabled.**
- 2) **Make sure the phone Bluetooth is enabled.**
- 3) **Run the App**, then select the Device ID for the initial use.

Note: DO NOT PAIR in the settings of your smartphone.

2.10 Add a new device

For the initial use, you need to add a new device.

- 1) Turn on device, run App, select <Checkme O2>;

- 2) Then input the **last 4 digits of SN** on the back of your device main unit.

2.11 Real-time Dashboard

You can check real-time SpO₂, Pulse Rate in App.

- 1) Connect device to App.
- 2) App->Dashboard.

2.12 Smart Vibration

The vibrator in the sensor will be activated when the SpO₂ fall below the pre-set value (Threshold). The vibration will stop when the SpO₂ recover, or you can press the button to stop it.


You can customize the vibration to help you sleep better. Make sure your **device is connected** to App first. Then you can configure it in **App->Settings**.

- You can **switch on or off** the vibration.
- You can adjust the **Intensity** of vibration.
- You can adjust the **Threshold**. If the vibration disturbs you during sleep too much, you can lower the **Threshold**. If you would like to get more protection from oxygen shortage, you can turn up the **Threshold**.

2.13 View Results and Report

In App, you can review all the history data. Touch a record, there comes out the detailed report, which includes analysis results and charts.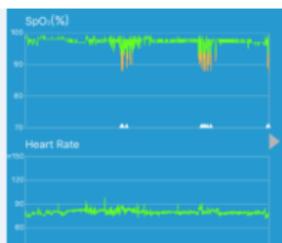

Slide a record to left, you can delete it.

2.14 O2 Score

O2 Score is overall assessment of oxygen condition, which synthesizes the frequency, depth and duration of oxygen shortage overnight. The range is 0-10 (10 is best). It is provided for each record in App.

Example:

3 Maintenance

3.1 Cleaning

Use a soft cloth moistened with water or alcohol to clean the device surface.

3.2 Battery

To keep the battery in good condition, charge the battery every 6 months when the device is not in use.

4 Troubleshooting

Problem	Possible Cause	Possible Solution
Device does not turn on or no response.	Battery may be low.	Charge battery and try again.
	Unexpected software condition	Press the button for about 10 seconds to reset
	Device might be damaged.	Please contact your local distributor.

The app cannot find the device.	The Bluetooth of your phone is off.	Turn on the Bluetooth in the phone.
	The device Bluetooth is off in Sleep Mode.	Press the button, the Bluetooth will be turned on when the screen lights up.

5 Specifications

Classifications		
EC Directive	MDD, 93/42/EEC	
	R&TTE, 1999/5/EC	
	ROHS 2.0, 2011/65/EU	
Degree protection against electrical shock	Type BF	
Environmental		
Item	Operating	Storage
Temperature	5 to 40°C	-25 to 70°C
Relative humidity (noncondensing)	10% to 95%	10% to 95%
Barometric	700 to 1060 hPa	700 to 1060 hPa
Degree of dust & water resistance	IP22	
Physical		
Weight	12 g (main unit)	
Display	OLED	
Wireless	Bluetooth 4.0 BLE	
Vibrator	Built in	
Power Supply		
Charge input:	DC 5V ±10%	
Battery type	Rechargeable lithium-polymer battery	
Charge time	2~3 hours	
SpO₂		
Standards	Meet standards of ISO 80601-2-61	
Measurement accuracy verification: The SpO ₂ accuracy has been verified in human experiments by comparing with arterial blood		

sample reference measured with a CO-oximeter. The pulse rate accuracy has been verified by Emulator. Pulse oximeter measurement are statistically distributed and about two-thirds of the measurements are expected to come within the specified accuracy range compared to CO-oximeter measurements.	
SpO ₂ range	70% to 100%
SpO ₂ Accuracy (Arms)	80-100%:±2%, 70-79%:±3%
PR range	30 to 250 bpm
PR accuracy	±2 bpm or ±2%, whichever is greater
Wave length	660-940nm
Output power	Red/Infrared: 3mW max. avg.
Pedometer	
Range	0 to 99999 steps
Storage	
Capacity	4 records,10 hours for each
Sleep analysis	
Record parameters	SpO ₂ , pulse rate, motion
Record interval	4s
Drop analysis	Yes
Mobile APP	
iOS	iOS 9.0 or above, iPhone 4s/iPad 3 or above
Android	Android 5.0 or above, with <i>Bluetooth</i> 4.0 BLE

6 Electromagnetic Compatibility

The device meets the requirements of EN 60601-1-2.

⚠ Warnings and Cautions

- Using accessories other than those specified in this manual may result in increased electromagnetic emission or decreased electromagnetic immunity of the equipment.
- The device or its components should not be used adjacent to or stacked

with other equipment.

- The device needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided below.
- Other devices may interfere with this device even though they meet the requirements of CISPR.
- When the inputted signal is below the minimum amplitude provided in technical specifications, erroneous measurements could result.
- Portable and mobile communication equipment may affect the performance of this device.
- Other devices that have RF transmitter or source may affect this device (e.g. cell phones, PDAs, and PCs with wireless function).

EMC table information is listed on our website:

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©Copyright 2016 Shenzhen Viatom Technology Co., Ltd. All rights reserved.
Shenzhen Viatom Technology Co., Ltd
MedNet GmbH