PEAK FLOW METER

CE 0120

HS CLEMENT CLARKE INTERNATIONAL

@ YOUR PEAK FLOW METER
Your Mini-Wright Peak Flow Meter will help you to monitor your asthma by measuring your peak flow. This leaflet explains how to use your Peak Flow Meter, and how to record the results. With careful use the Mini-Wright would be expected to last for three years.

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
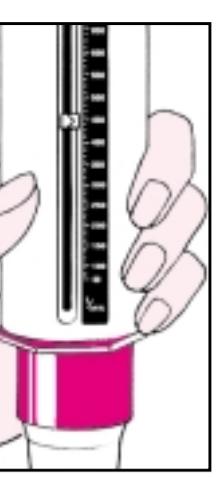
2 **Insert the mouthpiece into the meter, if not already fitted. Ensure the pointer is set at Zero (L/MIN position).**


3 **Stand up if possible. Take a deep breath, place the Peak Flow Meter in the mouth and hold horizontally, closing the lips around the mouthpiece, then blow as hard and as fast as you can.**

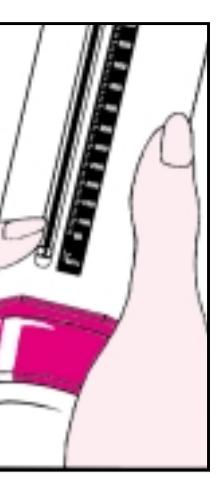4 **Note the number on the scale indicated by the pointer.**

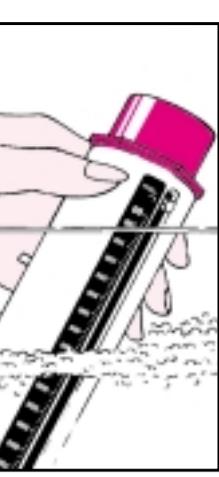5 **Return the pointer to zero (L/MIN position) and repeat the procedure twice more to obtain three readings. Read the highest of the three readings on the chart as shown overleaf.**

6 **CLEANING YOUR PEAK FLOW METER**

Immerse Peak Flow Meter in warm (but not hot) mild detergent solution for 2-3 minutes (maximum 5 minutes). Agitate the meter to ensure thorough cleaning.

Rinse in clean warm water and shake gently to remove any excess water. Allow to dry thoroughly before using again.

Standard Peak Flow Meter

① Standard Peak Flow Meter complete with detachable Universal Mouthpiece and Instructions/Recording Chart

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Instruction Leaflet. Cat. No. 1902169. Issue 4. 02/01.

④ CARE AND USE OF YOUR PEAK FLOW METER
It is important to keep your Peak Flow Meter clean when in use. It should be cleaned at once every six months and thoroughly if it has not been used for some time. The mouthpiece is detachable and should be disinfected before each use using a proprietary disinfectant solution such as Dettol. The Mini-Wright Peak Flow Meter is designed for single patient use. For healthcare professionals, cleaning instructions for multi patient use can be obtained from Clement Clarke International Limited. See address overleaf. Fungal growths may occur if the instrument has not been cleaned for some time.

PERFORMANCE ACCURACY The Mini-Wright standard range Peak Flow Meter meets NAEP guidelines and UK Department of Health specification no. 51.

If you have received a Black Scale meter: variations <+10% with Standard Wright Peak Flow Meter.

If you have received a Purple Scale meter, stating "ATS 94": Variations <±5% or 10 L/min (whichever is the greater) on ATS waveform generator PW24 multiples.

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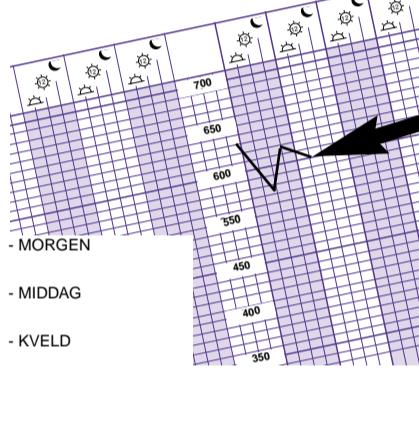
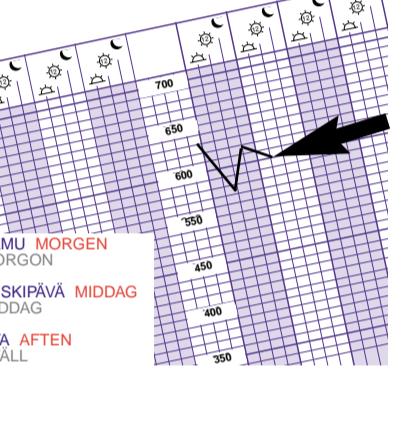
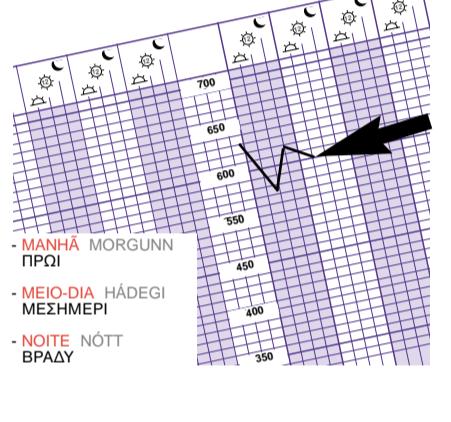
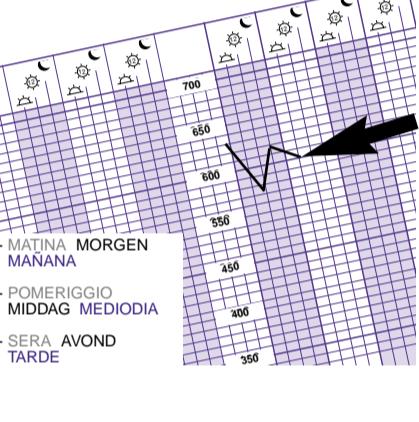
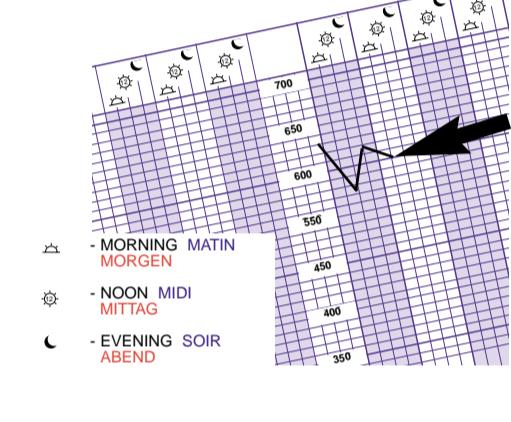
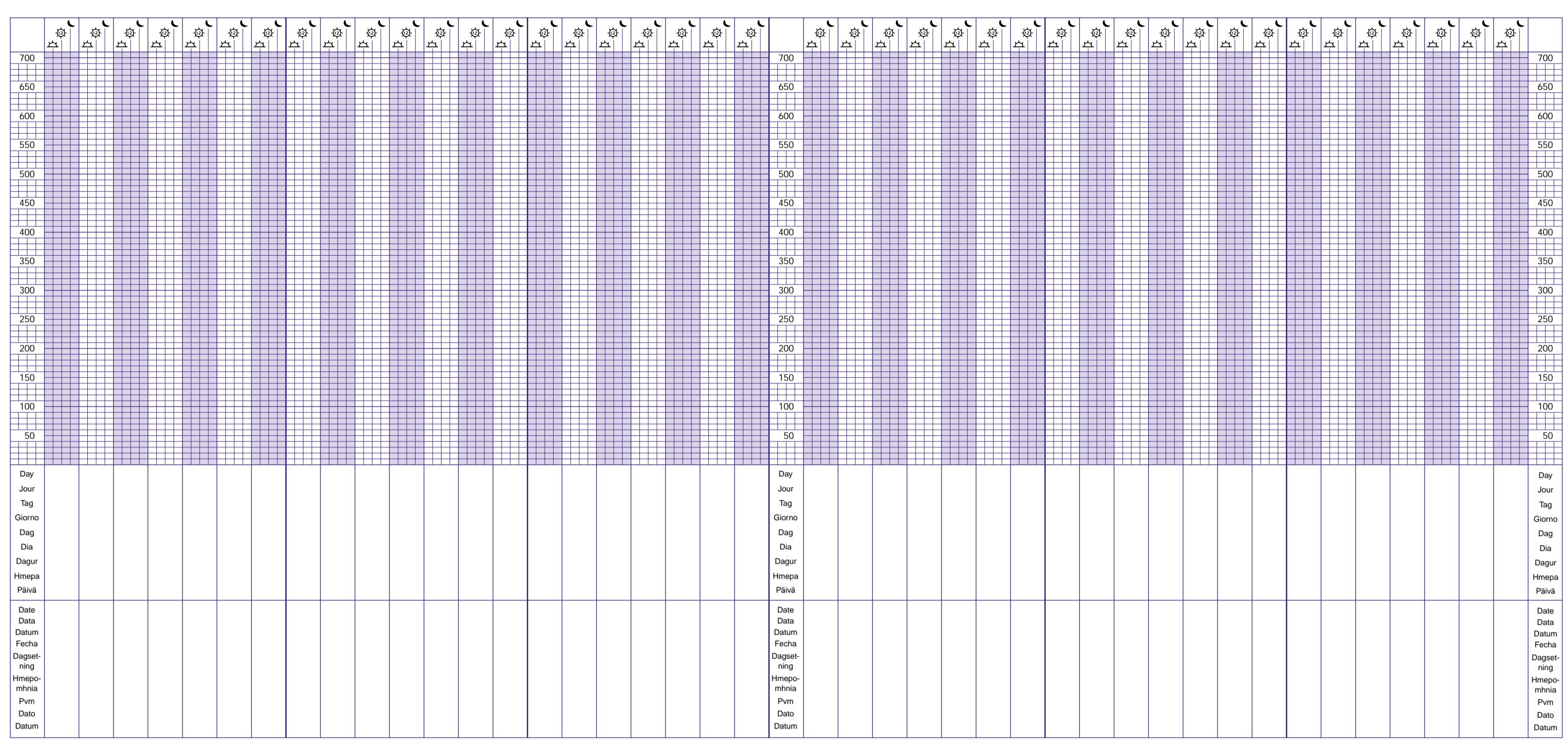
Date
Data
Datum
Fecha
Dagset-
ning
Hmepo-
mnia
Pvm
Dato
Datum

Date
Data
Datum
Fecha
Dagset-
ning
Hmepo-
mnia
Pvm
Dato
Datum

HOW TO RECORD YOUR PEAK FLOW
It is important to keep a record of your peak flow. Readings should be taken twice a day, first thing in the morning, and at bed time, or as directed by your physician. Use the chart provided to record the highest reading obtained each time. Mark your reading on the chart with a dot corresponding to the highest reading obtained for each of the morning, and bed time readings every day. By joining the dots (see diagram) you can see how your peak flow varies from day to day. You should also measure and note your peak flow if you suddenly become wheezy or breathless at any time of the day or night.

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