Medical Devices

Baby Mask

MANUALE ISTRUZIONI D'USO - Mascherine in silicone con raccordo
INSTRUCTIONS FOR USE MANUAL - Small silicone mask with coupling
MODE D'EMPLOI - Masques en silicone avec raccord
GEbruiksaanwijzing - Siliconen masker met opzetstuk
BEDIENUNGSANLEITUNG - Silikonschutzmasken mit Anschluss
MANUAL DE INSTRUCCIONES - Mascarillas de silicona con enganche
Руководство по эксплуатации - Силиконовые маски с соединением

**PHthalATES
& BPA FREE**

 **50** 
years of italian excellence

ITALIANO
Pag. 1

ENGLISH
pg. 5

FRANÇAIS
Pag. 9

NEDERLANDS
Pag. 13

DEUTSCH
Pag. 17

ESPAÑOL
PÁG. 21

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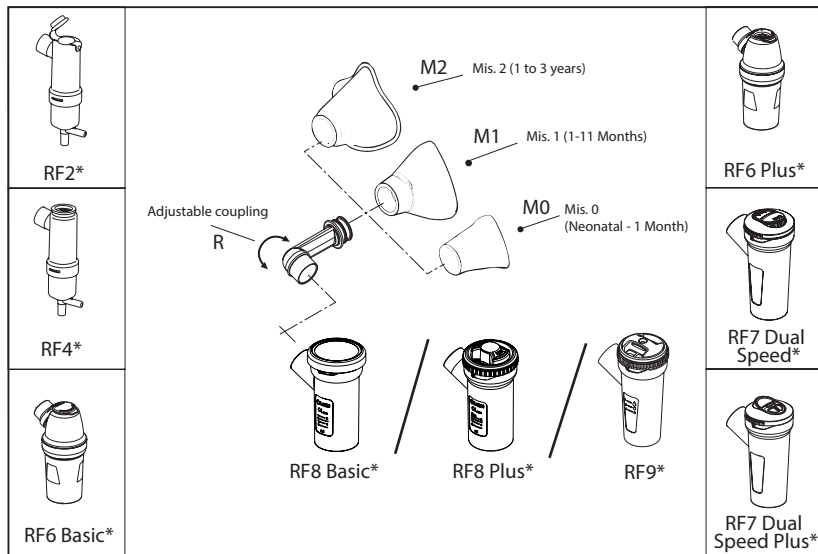
Baby Mask

Neonatal Set

Congratulations on your purchase and thank you for choosing our unit. Our goal is to fully satisfy consumers by offering them cutting-edge systems for the treatment of respiratory tract ailments. Carefully read these instructions and keep them in a safe place for future reference. Please visit **www.flaem.it** to see the entire range of Flaem products.

ENGLISH

ASSEMBLY DIAGRAM

* Optional accessories sold separately

INTENDED USE

The Neonatal Set supplies therapy to infants and children who still are not able to perform inhalation with a standard size mouthpiece or mask.

IMPORTANT SAFEGUARDS

- This is an accessories (compliant with Dir. 93/42/EEC) and may only be used with medication prescribed or recommended by a doctor. Before using the accessories, consult the instruction manual. It is important that the patient reads and understands the information on how to use the device and the nebulizer. Contact your retailer or nearest service center for any questions.
- The expected medical life of the accessories is 1 year. It is, however, advisable to replace the accessories every 6 months in the event of intense use to always guarantee maximum therapeutic efficacy.
- Children and disabled persons should always use the accessories under strict supervision of an adult who has read this manual.

INSTRUCTION FOR USE

Before and after each use, the accessories should be cleaned according to the instructions in the section **"CLEANING SANITISATION DISINFECTION STERILISATION"**.

A personal use of the accessories is recommended in order to avoid any risks of infection.

1. Wash your hands thoroughly before you start preparing the inhalation therapy.
2. Connect the accessories as shown in the "Assembly diagram"
3. Adjust the position of the adjustable coupling based on the position of the infant; proceed with treatment by activating the device as described in the user manual, keeping the beaker in a vertical position.

CLEANING SANITISATION DISINFECTION STERILISATION

CLEANING AT HOME - SANITISATION AND DISINFECTION

SANITIZATION: The accessories that can be sanitized are (M0,M1,M2,R).

Before and after each use, sanitise accessories, choosing one of the methods described below.

method A: Sanitise accessories under potable hot water (approximately 40°C) with a gentle, non abrasive dish detergent.

method B: Sanitise accessories in the dishwasher with a hot cycle.

method C: Sanitise accessories by immersing them in a solution of 50% water and 50% white vinegar, then rinse thoroughly under potable hot water (approximately 40°C).

If you want to also perform the cleaning for DISINFECTION, jump to the DISINFECTION paragraph.

After having sanitised the accessories, shake them vigorously and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example, a hair dryer).

DISINFECTION: The accessories that can be disinfected are (M0,M1,M2,R)

After sanitising the accessories, disinfect them choosing one of the methods described below.

method A: The disinfectant must be an electrolytic chloroxidizer (active principle: sodium hypochlorite) specific for disinfecting, which is available in any pharmacy.

Implementation:

- Fill a container big enough to hold all of the parts to disinfect with a solution of potable water and disinfectant, according to the proportions indicated on the packaging of the disinfectant.
- Completely immerse each part in the solution, taking care to avoid the formation of air bubbles on the parts. Leave the parts immersed for the amount of time indicated on the packaging of the disinfectant associated with the concentration chosen for the solution.
- Remove the disinfected parts and rinse abundantly with warm potable water.
- Dispose of the solution following the instructions provided by the disinfectant manufacturer.

method B: Disinfect the accessories by boiling them in water for 10 minutes; use demineralised or distilled water to prevent calcium deposits.

method C: Disinfect the accessories with a hot steam steriliser for baby-bottle (not the microwave type). Perform the process faithfully following the instructions of the steriliser. To ensure that the disinfection is effective, choose a steriliser with an operating cycle of at least 6 minutes.

After having disinfected the accessories, shake them vigorously and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example, from a hair dryer).

CLEANING IN A CLINICAL OR HOSPITAL SETTING - DISINFECTION AND STERILISATION

SANITIZATION: The accessories that can be sanitized are (M0,M1,M2,R).

Before disinfection or sterilisation, sanitise the accessories, choosing one of the methods described below.

method A: Sanitise accessories under potable hot water (approximately 40°C) with a gentle, non abrasive dish detergent.

method B: Sanitise accessories in the dishwasher with a hot cycle.

DISINFECTION

DISINFECTION: The accessories that can be disinfected are (M0,M1,M2,R)

The disinfectant must be an electrolytic chloroxidizer (active principle: sodium hypochlorite) specific for disinfecting, which is available in any pharmacy.

Implementation:

- Fill a container big enough to hold all of the parts to disinfect with a solution of potable water and disinfectant, according to the proportions indicated on the packaging of the disinfectant.
 - Completely immerse each part in the solution, taking care to avoid the formation of air bubbles on the parts. Leave the parts immersed for the amount of time indicated on the packaging of the disinfectant associated with the concentration chosen for the solution.
 - Remove the disinfected parts and rinse abundantly with warm potable water.
 - Dispose of the solution following the instructions provided by the disinfectant manufacturer.
- If you want to also perform the STERILISATION, jump to the STERILISATION paragraph.

After having disinfected the accessories, shake them vigorously and place them on a paper

towel. Alternatively, dry them with a jet of hot air (for example, from a hair dryer).

STERILIZATION: The accessories that can be sterilized are (M0,M1,M2,R)

Device: Fractionated vacuum overpressure steam steriliser in accordance with EN 13060.

Implementation: Wrap every single part to be treated with a sterile barrier system or packaging in accordance with Norm EN 11607. Place the packed components in the steam steriliser. Run the sterilisation cycle according to the operating instructions of the device by selecting a temperature of 134°C and a time of 10 minutes first.

Storage: Store the sterilised parts as per the instructions for use of either the sterile barrier system or packaging.

The sterilisation procedure is validated in its conformity to ISO 17665-1.

At the end of each use store the accessories in a dry place away from dust.

INTERACTIONS

The materials used that come into contact with medications have been tested with a wide range of medications. However, due to the variety and continuous evolution of medication, the possibility of a chemical interaction cannot be excluded. We recommend that once the medication has been opened you should use it as soon as possible and avoid prolonged contact of the medication with the nebulizer.

Use only Flaem originals accessories.

MODEL: M0= B014 - M1=B015 - M2=B016 - R=B002 **The applied parts of type BF are:** (M0, M1, M2)

SYMBOLS

	EC Marking medical ref. Dir 93/42 EEC and subsequent updates		Check the instructions for use
	In compliance with: European Standard EN 10993-1 "Biological Evaluation of medical devices" and European Directive 93/42/EEC "Medical Devices." Phthalate-free. In compliance with: Reg. (EC) no. 1907/2006		Manufacturer
			Lot Number

PACKAGING DISPOSAL

Product packaging bag and Tube packaging bag

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FLAEM NUOVA S.p.A. non risponde di ulteriori estensioni del periodo di garanzia assicurate da terzi.
FLAEM NUOVA S.p.A. - Via Colli Storic, 221