8	124	0.46
6	80	0.37
L/min	Kpa	g/min

**PHILIPS**  
**RESPIRONICS**

SideStream  
disposable kit

www.philips.com/respironics

REF 4446, 4446A

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**Mask** - Wipe mask with a clean dry cloth and allow to air dry.

- Warning:** Do not immerse tubing in water.
- Reassemble making sure the baffle is in the correct position.
- Shake out excess water and allow to air dry.
- Rinse all parts in clean, cold water.

- Take apart your SideStream top, medicine cup, baffle and mouthpiece and wash in warm, soapy water.
- Take apart your SideStream top, medicine cup, baffle and mouthpiece and wash in warm, soapy water.

**Cleaning** - After each use:

- Recommended fill volume 2 – 10 ml.
- Use of 22mm adapters may reduce output rate.
- Do not lay your SideStream on its side when full of drug.
- Do not remove the top of your SideStream when connected to an air supply.

- Do not remove the top of your SideStream when cleaning it because this will damage the nebulizer.
- Do not dry your SideStream with a towel. Never use a cleaning brush or put sharp objects into the jet holes or baffle when cleaning it because this will damage the nebulizer.
- For single patient use.

**Caution**

- or via www.philips.com/sidestream
- Aerosol data according to EN13544-1 is available on request from Philips Respironics

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